



iTERO IMPLANT SCAN
BODY CERTIFIED
CONNECTIVITY WORKFLOWS




CREATING IMPLANT SCAN BODY FAVORITES


Follow the steps below to customize the list of scan bodies in MyAligntech and on the iTero scanner. The list created will be referred to as a 'favorites' list. Once selected and saved, implant scan body favorites will be stored in MyAligntech and on the iTero scanner.

STEP 1

To create the favorites first log in to MyAligntech.

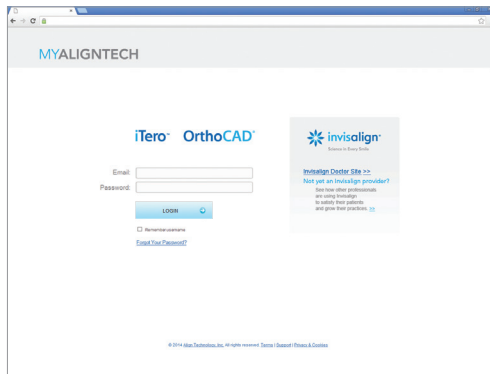


Click on the "Profile" link at the top right of the page.

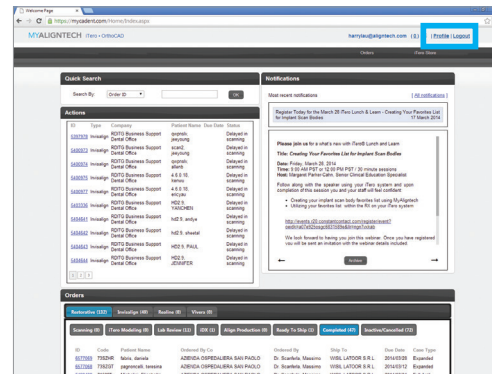


STEP 2

Click the "Manage Favorites" link under the Profile section.

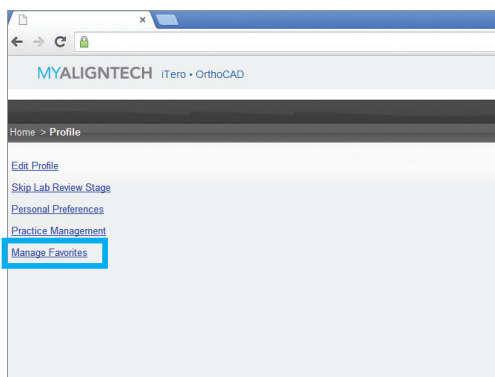



STEP 3

The description begins with the name of the scan body manufacturer, followed by the implant manufacturer, and then the specific scan body/implant configuration. Select the applicable configurations to meet your clinical needs on the left side master list. Add them to your favorites list by clicking on the arrow pointing to the favorites list. Click Save prior to exiting.

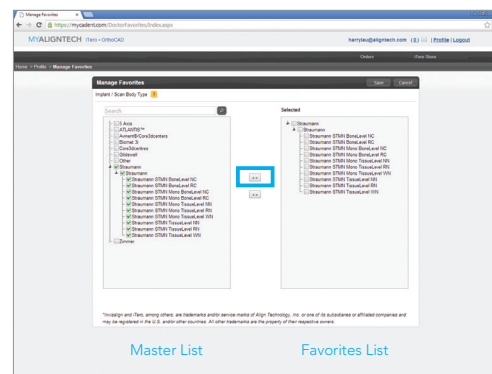

STEP 4

After restarting your iTero scanner the new favorites list should be available in the RX information page. To manually update the configuration on your iTero scanner the following sequence can be used from the iTero menu toolbar: Menu > File > Tools > Update Configuration.

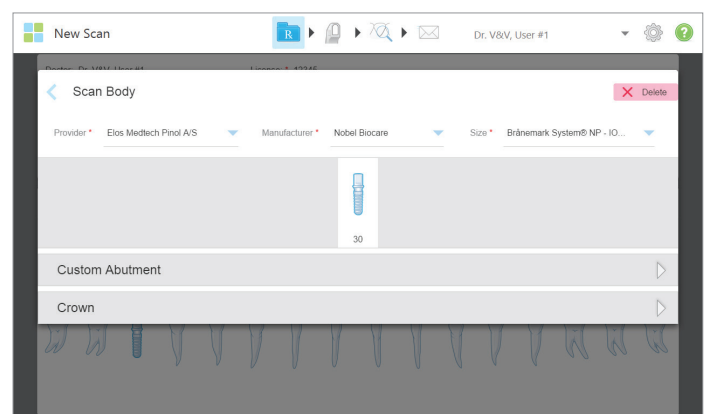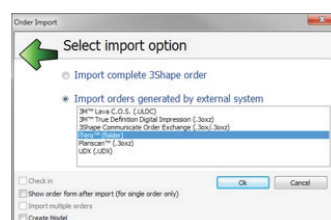

IMPLANT SCAN BODY CONNECTED WORKFLOW VENDORS

iTero® Certified Connectivity with implant scan body manufacturers offer clinicians a wide range of scan body and implant abutment manufacturer choices. Clinicians can utilize iTero to scan the scan body of their choice and send to an iTero Certified Connectivity implant vendor.



5 AXIS DENTAL

Ontario, Canada | www.5axisdental.com | 1-855-MY 5 AXIS (855-695-2947) Toll Free 905-665-9930

Scan Body Manufacturer	Scan Body ID	Implant Manufacturer	Scan Body / Implant Type
5 Axis Dental	BioH-301	BioHorizons®	5 Axis Dental BioHorizons Internal 3.0 Hex [BioH-301]
5 Axis Dental	BioH-351	BioHorizons®	5 Axis Dental BioHorizons Internal 4.0/4.5 Hex [BioH-451]
5 Axis Dental	BioH-571	BioHorizons®	5 Axis Dental BioHorizons Internal 5.7 Hex [BioH-571]
5 Axis Dental	3i-341	Biomet 3i	5 Axis Dental Biomet 3i Internal 3.25/3.4 Hex [3i-341]
5 Axis Dental	3i-411	Biomet 3i	5 Axis Dental Biomet 3i Internal 4.0/4.1 Hex [3i-411]
5 Axis Dental	3i-501	Biomet 3i	5 Axis Dental Biomet 3i Internal 5.0 Hex [3i-501]
5 Axis Dental	3i-601	Biomet 3i	5 Axis Dental Biomet 3i Internal 6.0 Hex [3i-601]
5 Axis Dental	AST-3540	DENTSPLY Implants	5 Axis Dental Dentsply OsseoSpeed Internal 3.5/4.0 [AST-3540]
5 Axis Dental	AST-4550	DENTSPLY Implants	5 Axis Dental Dentsply OsseoSpeed Internal 4.5/5.0 [AST-4550]
5 Axis Dental	HIO-M	Hiossen	5 Axis Dental Hiossen Mini Internal 3.5 Hex [HIO-M] 5 Axis Dental
5 Axis Dental	HIT-IL	HI-TEC	5 Axis Dental Hiossen Regular Internal 4.0/4.5/5.0 Hex [HIO-Reg]
5 Axis Dental	HIT-WIL	HI-TEC	5 Axis Dental HI-TEC Internal Self Thread 4.2/4.5 Hex [HIT-WIL]
5 Axis Dental	HIT-IL	HI-TEC	5 Axis Dental HI-TEC Logic Plus Internal 3.5/4.3/5.0/6.0 Hex [HIT-IL]
5 Axis Dental	MIS-NP	Mis	5 Axis Dental MIS Seven Internal 3.0 Hex [MIS-NP]
5 Axis Dental	MIS-SP	Mis	5 Axis Dental MIS Seven Internal 3.75/4.2 SP Internal Hex [MIS-SP]
5 Axis Dental	MIS-WP	Mis	5 Axis Dental MIS Seven Internal 5.0/6.0 WP Internal Hex [MIS-WP]
5 Axis Dental	ACT-NP	Nobel Biocare	5 Axis Dental Nobel Active Internal 3.5 NP Conical Connection [ACT-NP]
5 Axis Dental	ACT-RP	Nobel Biocare	5 Axis Dental Nobel Active Internal 4.5/5.0 RP Conical Connection [ACT-RP]
5 Axis Dental	REP-NP	Nobel Biocare	5 Axis Dental Nobel Replace Internal 3.5 NP Trilobe [REP-NP]
5 Axis Dental	REP-RP	Nobel Biocare	5 Axis Dental Nobel Replace Internal 4.5 RP Trilobe [REP-RP]
5 Axis Dental	REP-WP	Nobel Biocare	5 Axis Dental Nobel Replace Internal 5.0 WP Trilobe [REP-WP]
5 Axis Dental	REP-60	Nobel Biocare	5 Axis Dental Nobel Replace Internal 6.0 Trilobe [REP-60]
5 Axis Dental	REP-RP	Southern	5 Axis Dental Southern Implants Tri-Nex Co-Axis Internal 5.0 Trilobe [REP-RP]
5 Axis Dental	REP-NP	Southern	5 Axis Dental Southern Implants Tri-Nex Internal 3.5 NP Trilobe [REP-NP]
5 Axis Dental	REP-RP	Southern	5 Axis Dental Southern Implants Tri-Nex Internal 4.3 RP Trilobe [REP-RP]
5 Axis Dental	REP-WP	Southern	5 Axis Dental Southern Implants Tri-Nex Internal 5.0/7.0 WP Trilobe [REP-WP]
5 Axis Dental	REP-60	Southern	5 Axis Dental Southern Implants Tri-Nex Internal 6.0/8.0/9.0 Trilobe [REP-60]
5 Axis Dental	REP-NP	Southern	5 Axis Dental Southern Implants Tri-Nex Internal Co-Axis 4.3 Trilobe [REP-NP]
5 Axis Dental	STR-TLRN	Southern	5 Axis Dental Southern MAX-IT Internal 7.0 Octagonal [STR-TLRN]
5 Axis Dental	STR-TLWN	Southern	5 Axis Dental Southern MAX-IT Internal 8.0/9.0 Octagonal [STR-TLWN]
5 Axis Dental	STR-TLRN	Southern	5 Axis Dental Southern OCT-XP Internal 3.3/4.1/4.9 Octagonal [STR-TLRN]
5 Axis Dental	STR_TLRN	Southern	5 Axis Dental Southern OCT-XT Internal 4.0/5.0 Octagonal [STR-TLRN]
5 Axis Dental	STR-TLRN	Southern	5 Axis Dental Southern OCT-XT Internal 5.0 Co-Axis Octagonal [STR-TLRN]
5 Axis Dental	STR-TLWN	Southern	5 Axis Dental Southern OCT-XT Internal 6.0 Octagonal [STR-TLWN]
5 Axis Dental	STR-BLNC	Straumann	5 Axis Dental Straumann Bone Level Internal 3.3 NC Octagonal [STR-BLNC]
5 Axis Dental	STR-BLRC	Straumann	5 Axis Dental Straumann Bone Level Internal 4.1/4.8 RC Octagonal [STR-BLRC]
5 Axis Dental	STR-TLNN	Straumann	5 Axis Dental Straumann Tissue Level Internal 3.5 NN Octagonal [STR-TLNN]
5 Axis Dental	STR-TLRN	Straumann	5 Axis Dental Straumann Tissue Level Internal 4.3 RN Octagonal [STR-TLRN]
5 Axis Dental	STR-TLWN	Straumann	5 Axis Dental Straumann Tissue Level Internal 5.0 WN Octagonal [STR-TLWN]

Scan Body Manufacturer	Scan Body ID	Implant Manufacturer	Scan Body / Implant Type
ATLANTIS™	IO-B-01	BioHorizons®	Internal, Tapered Internal 3.5 [IO-B-01] [110]
ATLANTIS™	IO-B-02	BioHorizons®	Internal, Tapered Internal 4.5 [IO-B-02] [111]
ATLANTIS™	IO-B-03	BioHorizons®	Internal, Tapered Internal 5.7 [IO-B-03] [112]
ATLANTIS™	IO-F-01	Biomet 3i	ATLANTIS™ Biomet 3i Certain™ 3.25 [IO-F-01] [62]
ATLANTIS™	IO-F-01	Biomet 3i	Certain™ 3.25 [IO-F-01] [62]
ATLANTIS™	IO-F-02	Biomet 3i	Certain™ 4.0 [IO-F-02] [8]
ATLANTIS™	IO-F-01	Biomet 3i	Certain™ 4/3 [IO-F-01] [119]
ATLANTIS™	IO-F-02	Biomet 3i	Certain™ 5.0 [IO-F-02] [9]
ATLANTIS™	IO-F-02	Biomet 3i	Certain™ 5/4 [IO-F-02] [120]
ATLANTIS™	IO-F-02	Biomet 3i	Certain™ 6.0 [IO-F-02] [10]
ATLANTIS™	IO-F-01	Biomet 3i	Certain™ Prevail™ 3/4/3 [IO-F-01] [71]
ATLANTIS™	IO-F-01	Biomet 3i	Certain™ Prevail™ 4/3 [IO-F-01] [72]
ATLANTIS™	IO-F-02	Biomet 3i	Certain™ Prevail™ 4/5/4 [IO-F-02] [73]
ATLANTIS™	IO-F-02	Biomet 3i	Certain™ Prevail™ 5/4 [IO-F-02] [74]
ATLANTIS™	IO-F-02	Biomet 3i	Certain™ Prevail™ 5/6/5 [IO-F-02] [76]
ATLANTIS™	IO-F-02	Biomet 3i	Certain™ Prevail™ 6/5 [IO-F-02] [77]
ATLANTIS™	IO-F-02	Biomet 3i	Certain™ XP 4/5 [IO-F-02] [75]
ATLANTIS™	IO-F-02	Biomet 3i	Certain™ XP 5/6 [IO-F-02] [78]
ATLANTIS™	IO-N-02	CAMLOG	ATLANTIS™ Camlog Root-line Implant 3.8 [IO-N-02] [136]
ATLANTIS™	IO-N-03	CAMLOG	ATLANTIS™ Camlog Root-line Implant 4.3 [IO-N-03] [137]
ATLANTIS™	IO-N-04	CAMLOG	ATLANTIS™ Camlog Root-line Implant 5.0 [IO-N-04] [138]
ATLANTIS™	IO-N-04	CAMLOG	ATLANTIS™ Camlog Root-line Implant 6.0 [IO-N-04] [139]
ATLANTIS™	IO-N-02	CAMLOG	ATLANTIS™ Camlog Screw-line [IO-N-02] [132]
ATLANTIS™	IO-N-03	CAMLOG	ATLANTIS™ Camlog Screw-line [IO-N-03] [133]
ATLANTIS™	IO-N-01	CAMLOG	ATLANTIS™ Camlog Screw-line Implant 3.3 [IO-N-01] [130]10249
ATLANTIS™	IO-N-04	CAMLOG	ATLANTIS™ Camlog Screw-line Implant 5.0 [IO-N-04] [134]
ATLANTIS™	IO-N-04	CAMLOG	ATLANTIS™ Camlog Screw-line Implant 6.0 [IO-N-04] [138]
ATLANTIS™	IO-J-03	DENTSPLY Implants	ATLANTIS™ DENTSPLY XiVE® S 3.8 [IO-J-03] [88]
ATLANTIS™	IO-J-04	DENTSPLY Implants	ATLANTIS™ DENTSPLY XiVE® S 4.5 [IO-J-04] [89]
ATLANTIS™	IO-J-05	DENTSPLY Implants	ATLANTIS™ DENTSPLY XiVE® S 5.5 [IO-J-05] [90]
ATLANTIS™	IO-P-06	DENTSPLY Implants	ATLANTIS™ DENTSPLY OsseoSpeed™ Profile EV 4.2 [IO-P-06] [174]
ATLANTIS™	IO-P-07	DENTSPLY Implants	ATLANTIS™ DENTSPLY OsseoSpeed™ Profile EV 4.8 [IO-P-07] [173]
ATLANTIS™	IO-C-04	DENTSPLY Implants	ATLANTIS™ DENTSPLY OsseoSpeed™ TX Profile 4.5, 5.0 [IO-C-04] [126]
ATLANTIS™	IO-L-01	DENTSPLY Implants	ATLANTIS™ DENTSPLY ANKYLOS® C/X 3.5 [IO-L-01] [95]
ATLANTIS™	IO-L-01	DENTSPLY Implants	ATLANTIS™ DENTSPLY ANKYLOS® C/X 4.5 [IO-L-01] [96]
ATLANTIS™	IO-L-01	DENTSPLY Implants	ATLANTIS™ DENTSPLY ANKYLOS® C/X 5.5 [IO-L-01] [97]
ATLANTIS™	IO-L-01	DENTSPLY Implants	ATLANTIS™ DENTSPLY ANKYLOS® C/X 7.0 [IO-L-01] [98]
ATLANTIS™	IO-J-02	DENTSPLY Implants	Frialit™ 3.4 [IO-J-02] [100]
ATLANTIS™	IO-C-01	DENTSPLY Implants	OsseoSpeed™ 3.0 – Yellow [IO-C-01] [70]
ATLANTIS™	IO-C-02	DENTSPLY Implants	OsseoSpeed™ 3.5, 4.0 – Aqua [IO-C-02] [64]
ATLANTIS™	IO-C-03	DENTSPLY Implants	OsseoSpeed™ 4.5, 5.0 – Lilac [IO-C-03] [66]
ATLANTIS™	IO-P-01	DENTSPLY Implants	OsseoSpeed™ EV 3.0 - Green [IO-P-01] [148]
ATLANTIS™	IO-P-02	DENTSPLY Implants	OsseoSpeed™ EV 3.6 - Purple [IO-P-02] [140]
ATLANTIS™	IO-P-03	DENTSPLY Implants	OsseoSpeed™ EV 4.2 - Yellow [IO-P-03] [141]
ATLANTIS™	IO-P-04	DENTSPLY Implants	OsseoSpeed™ EV 4.8 - Blue [IO-P-04] [146]
ATLANTIS™	IO-P-05	DENTSPLY Implants	OsseoSpeed™ EV 5.4 - Brown [IO-P-05] [147]
ATLANTIS™	IO-C-01	DENTSPLY Implants	OsseoSpeed™ TX 3.0 – Yellow [IO-C-01] [92]

ATLANTIS™ (continued)

DENTSPLY Implants | Waltham, MA | implants-na-cs@dentsply.com | 1-800-531-3481 or Fax 1-781-890-6808 USA

Scan Body Manufacturer	Scan Body ID	Implant Manufacturer	Scan Body / Implant Type
ATLANTIS™	IO-C-02	DENTSPLY Implants	OsseoSpeed™ TX 3.5, 4.0 – Aqua [IO-C-02] [93]
ATLANTIS™	IO-C-03	DENTSPLY Implants	OsseoSpeed™ TX 4.5, 5.0 – Lilac [IO-C-03] [94]
ATLANTIS™	IO-J-01	DENTSPLY Implants	XiVE® S 3.0 [IO-J-01] [86]
ATLANTIS™	IO-J-02	DENTSPLY Implants	XiVE® S 3.4 [IO-J-02] [87]
ATLANTIS™	IO-K-01	Nobel Biocare	ATLANTIS™ Nobel Biocare NobelActive™ / NobelReplace™ Conical C NP 3.5 [IO-K-01] [170]
ATLANTIS™	IO-K-02	Nobel Biocare	ATLANTIS™ Nobel Biocare NobelActive™ / NobelReplace™ Conical C RP 4.3 [IO-K-02] [171]
ATLANTIS™	IO-K-02	Nobel Biocare	ATLANTIS™ Nobel Biocare NobelActive™ / NobelReplace™ Conical C RP 5.0 [IO-K-02] [172]
ATLANTIS™	IO-D-01	Nobel Biocare	ATLANTIS™ Nobel Biocare NobelReplace™ NP 3.5 [IO-D-01] [36]
ATLANTIS™	IO-D-02	Nobel Biocare	ATLANTIS™ Nobel Biocare NobelReplace™ RP 4.3 [IO-D-02] [37]
ATLANTIS™	IO-D-03	Nobel Biocare	ATLANTIS™ Nobel Biocare NobelReplace™ WP 5.0 [IO-D-03] [38]
ATLANTIS™	IO-H-01	Straumann	ATLANTIS™ Straumann Bone Level 3.3 NC [IO-H-01] [79]
ATLANTIS™	IO-H-02	Straumann	ATLANTIS™ Straumann Bone Level 4.1 RC [IO-H-02] [80]
ATLANTIS™	IO-H-02	Straumann	ATLANTIS™ Straumann Bone Level 4.8 RC [IO-H-02] [82]
ATLANTIS™	IO-B-01	Zimmer	ATLANTIS™ Zimmer Screw-Vent™ 3.3 friction fit [IO-B-01] [159]
ATLANTIS™	IO-B-01	Zimmer	ATLANTIS™ Zimmer Screw-Vent™ 3.7 friction fit [IO-B-01] [160]
ATLANTIS™	IO-B-02	Zimmer	ATLANTIS™ Zimmer Screw-Vent™ 4.5 friction fit [IO-B-02] [161]
ATLANTIS™	IO-B-01	Zimmer	ATLANTIS™ Zimmer Tapered Screw-Vent™ 3.5 friction fit [IO-B-01] [156]
ATLANTIS™	IO-B-02	Zimmer	ATLANTIS™ Zimmer Tapered Screw-Vent™ 4.5 friction fit [IO-B-02] [157]
ATLANTIS™	IO-B-03	Zimmer	ATLANTIS™ Zimmer Tapered Screw-Vent™ 5.7 friction fit [IO-B-03] [158]
ATLANTIS™	IO-B-01	Zimmer	Screw Vent™ 3.3 [IO-B-01] [105]
ATLANTIS™	IO-B-01	Zimmer	Screw Vent™ 3.7 [IO-B-01] [106]
ATLANTIS™	IO-B-02	Zimmer	Screw Vent™ 4.5 [IO-B-02] [104]
ATLANTIS™	IO-B-01	Zimmer	Tapered Screw Vent™ 3.5 [IO-B-01] [47]
ATLANTIS™	IO-B-02	Zimmer	Tapered Screw Vent™ 4.5 [IO-B-02] [48]
ATLANTIS™	IO-B-03	Zimmer	Tapered Screw Vent™ 5.7 [IO-B-03] [49]

AVINENT™ / CORE3DCENTERS

Las Vegas, USA | www.core3dcentres.com/usa/ | 1-888-750-9204

Scan Body Manufacturer	Scan Body ID	Implant Manufacturer	Scan Body / Implant Type
Avinent® / Core3dcenters	1806	Avinent	Avinent® Implant System HE 3,5 Coral/Ocean [1806]
Avinent® / Core3dcenters	1807	Avinent	Avinent® Implant System HE 4,1 Coral/Ocean [1807]
Avinent® / Core3dcenters	1808	Avinent	Avinent® Implant System HE 5,1 Coral [1808]
Avinent® / Core3dcenters	1809	Avinent	Avinent® Implant System HI 3,5/4,1 Coral/Ocean [1809]
Avinent® / Core3dcenters	1811	Avinent	Avinent® Implant System Transeptial 4,8 Coral/Ocean [1811]
	50	Biomet 3i	Biomet 3i™ Encode
Core3dcentres	1806	Avinent	Core3dcentres Avinent HE 3,5 Coral/Ocean [1806]
Core3dcentres	1807	Avinent	Core3dcentres Avinent HE 4,1 Coral/Ocean [1807]
Core3dcentres	1808	Avinent	Core3dcentres Avinent HE 5,1 Coral [1808]
Core3dcentres	1809	Avinent	Core3dcentres Avinent HI 3,5/4,1 Coral/Ocean [1809]
Core3dcentres	1873	Bego Semados Implant Systems RI 4,5	Core3dcentres Bego Semados Implant Systems RI 4,5 [1873]
Core3dcentres	1873	Bego Semados Implant Systems S3,25-3,75-4.1 / RI 3,75-4.1	Core3dcentres Bego Semados Implant Systems S3,25-3,75-4.1 / RI 3,75-4.1 [1873]
Core3dcentres	1873	Bego Semados Implant Systems S5,5 / RI 5,5	Core3dcentres Bego Semados Implant Systems S5,5 / RI 5,5 [1873]

Scan Body Manufacturer	Scan Body ID	Implant Manufacturer	Scan Body / Implant Type
Core3dcentres	1646	BioComp	Core3dcentres BioComp [1646]
Core3dcentres	2713	BioHorizons*	Core3dcentres BioHorizons* Tapered Internal 3,0mm Gray [2713]
Core3dcentres	2625	BioHorizons*	Core3dcentres BioHorizons* Tapered Internal 3,5mm Yellow [2625]
Core3dcentres	2626	BioHorizons*	Core3dcentres BioHorizons* Tapered Internal 4,5mm Green [2626]
Core3dcentres	2627	BioHorizons*	Core3dcentres BioHorizons* Tapered Internal 5,7mm Blue [2627]
Core3dcentres	1834	Biomet 3i	Core3dcentres Biomet 3i Internal Certain* 6,0 [1834]
Core3dcentres	1830	Biomet 3i	Core3dcentres Biomet 3i™ External Connection 3,25 [1830]
Core3dcentres	2534	Biomet 3i	Core3dcentres Biomet 3i™ External Connection 4,0 [2534]
Core3dcentres	2534	Biomet 3i	Core3dcentres Biomet 3i™ External Connection 5,0 [2534]
Core3dcentres	2534	Biomet 3i	Core3dcentres Biomet 3i™ External Connection 6,0 [2534]
Core3dcentres	1833	Biomet 3i	Core3dcentres Biomet 3i™ Internal Certain* 3,4 [1833]
Core3dcentres	1834	Biomet 3i	Core3dcentres Biomet 3i™ Internal Certain* 4,1 [1834]
Core3dcentres	1834	Biomet 3i	Core3dcentres Biomet 3i™ Internal Certain* 5,0 [1834]
Core3dcentres	2195	BTI*	Core3dcentres BTI* Implante Externa* Ancha [2195]
Core3dcentres	2510	BTI*	Core3dcentres BTI* Implante Externa* Universal [2510]
Core3dcentres	2510	BTI*	Core3dcentres BTI* Implante Externa* Universal Plus [2510]
Core3dcentres	2196	BTI*	Core3dcentres BTI* Implante Interna* Ancha [2196]
Core3dcentres	2196	BTI*	Core3dcentres BTI* Implante Interna* Universal [2196]
Core3dcentres	2196	BTI*	Core3dcentres BTI* Implante Interna* Universal Plus [2196]
Core3dcentres	2190	BTI*	Core3dcentres BTI* Implante Tiny* [2190]
Core3dcentres	1832	CAMLOG	Core3dcentres CAMLOG Screw-Line Ø3,3 (grey) [1832]
Core3dcentres	1843	CAMLOG	Core3dcentres CAMLOG Screw-Line Ø3,8 (yellow) [1843]
Core3dcentres	1844	CAMLOG	Core3dcentres CAMLOG Screw-Line Ø4,3 (red) [1844]
Core3dcentres	1845	CAMLOG	Core3dcentres CAMLOG Screw-Line Ø5,0 (blue) [1845]
Core3dcentres	1845	CAMLOG	Core3dcentres CAMLOG Screw-Line Ø6,0 (green) [1845]
Core3dcentres	1768	Dentsply Friadent	Core3dcentres Dentsply Friadent Ankylos* C/X [1768]
Core3dcentres	2038	Dentsply Friadent	Core3dcentres Dentsply Friadent Xive* D3,0 (Brown) [2038]
Core3dcentres	2040	Dentsply Friadent	Core3dcentres Dentsply Friadent Xive* D3,4 (Grey) [2040]
Core3dcentres	2368	Dentsply Friadent	Core3dcentres Dentsply Friadent Xive* D3,8 (Yellow) [2368]
Core3dcentres	2368	Dentsply Friadent	Core3dcentres Dentsply Friadent Xive* D4,5 (Blue) [2368]
Core3dcentres	2368	Dentsply Friadent	Core3dcentres Dentsply Friadent Xive* D5,5 (Red) [2368]
Core3dcentres	2096	DENTSPLY Implants	Core3dcentres Dentsply OsseoSpeed™ 3,0 (Yellow) [2096]
Core3dcentres	1835	DENTSPLY Implants	Core3dcentres Dentsply OsseoSpeed™ 3,5/4,0 (Aqua) [1835]
Core3dcentres	1828	DENTSPLY Implants	Core3dcentres Dentsply OsseoSpeed™ 4,5/5,0 (Lilac) [1828]
Core3dcentres	2230	Keystone/Lifecore*	Core3dcentres Keystone/Lifecore* SD [2230]
Core3dcentres	2536	Keystone/Lifecore*	Core3dcentres Keystone/Lifecore* RD [2536]
Core3dcentres	2231	Keystone/Lifecore*	Core3dcentres Keystone/Lifecore* WD [2231]
Core3dcentres	2538	Microdent	Core3dcentres Microdent* Implant Microdent System Universal 4,1 [2538]
Core3dcentres	2538	Microdent	Core3dcentres Microdent* Implant Microdent System Universal 5,1 [2538]
Core3dcentres	1872	Mis	Core3dcentres Mis Internal Hexagon Narrow Platform [1872]
Core3dcentres	2628	Mis	Core3dcentres Mis Internal Hexagon Standard Platform [2628]
Core3dcentres	2629	Mis	Core3dcentres Mis Internal Hexagon Wide Platform [2629]
Core3dcentres	2078	Nobel Biocare	Core3dcentres Nobel Biocare Brånemark System™ NP [2078]
Core3dcentres	2093	Nobel Biocare	Core3dcentres Nobel Biocare Brånemark System™ RP [2093]
Core3dcentres	1781	Nobel Biocare	Core3dcentres Nobel Biocare Brånemark System™ WP [1781]
Core3dcentres	2845	Nobel Biocare	Core3dcentres Nobel Biocare Nobel Active™ 3.0 [2845]
Core3dcentres	2089	Nobel Biocare	Core3dcentres Nobel Biocare Nobel Active™ NP [2089]

AVINENT™ / CORE3DCENTERS (continued)

Las Vegas, USA | www.core3dcentres.com/usa/ | 1-888-750-9204

Scan Body Manufacturer	Scan Body ID	Implant Manufacturer	Scan Body / Implant Type
Core3dcentres	2090	Nobel Biocare	Core3dcentres Nobel Biocare Nobel Active™ RP [2090]
Core3dcentres	2084	Nobel Biocare	Core3dcentres Nobel Biocare Nobel Replace™ 6.0 [2084]
Core3dcentres	2081	Nobel Biocare	Core3dcentres Nobel Biocare Nobel Replace™ NP [2081]
Core3dcentres	2082	Nobel Biocare	Core3dcentres Nobel Biocare Nobel Replace™ RP [2082]
Core3dcentres	2083	Nobel Biocare	Core3dcentres Nobel Biocare Nobel Replace™ WP [2083]
Core3dcentres	2204	Simpler™	Core3dcentres Simpler™ implant solutions ST 5000 Series 3.25 [2204]
Core3dcentres	2215	Simpler™	Core3dcentres Simpler™ implant solutions ST 5000 Series 4.0 [2215]
Core3dcentres	2087	Straumann	Core3dcentres Straumann synOcta® Abutment RN 4,8 [2087]
Core3dcentres	2088	Straumann	Core3dcentres Straumann synOcta® Abutment WN 6,5 [2088]
Core3dcentres	2076	Straumann	Core3dcentres Straumann® NC [2076]
Core3dcentres	2077	Straumann	Core3dcentres Straumann® RC [2077]
Core3dcentres	2085	Straumann	Core3dcentres Straumann® RN [2085]
Core3dcentres	2086	Straumann	Core3dcentres Straumann® WN [2086]
Core3dcentres	1820	Tekka	Core3dcentres Tekka In-Kone® Universal 4.6 [1820]
Core3dcentres	1820	Tekka	Core3dcentres Tekka In-Kone® Universal 5.6 [1820]
Core3dcentres	1744	Tekka	Core3dcentres Tekka Progress 3.4 [1744]
Core3dcentres	1745	Tekka	Core3dcentres Tekka Progress 4 [1745]
Core3dcentres	1745	Tekka	Core3dcentres Tekka Progress 4.5 [1745]
Core3dcentres	1745	Tekka	Core3dcentres Tekka Progress 5 [1745]
Core3dcentres	1745	Tekka	Core3dcentres Tekka Progress 6 [1745]
Core3dcentres	2358	Thommen	Core3dcentres Thommen SPI® PF 3.5 [2358]
Core3dcentres	2359	Thommen	Core3dcentres Thommen SPI® PF 4 [2359]
Core3dcentres	2360	Thommen	Core3dcentres Thommen SPI® PF 4.5 [2360]
Core3dcentres	2361	Thommen	Core3dcentres Thommen SPI® PF 5 [2361]
Core3dcentres	2362	Thommen	Core3dcentres Thommen SPI® PF 6 [2362]
Core3dcentres	1786	Zimmer	Core3dcentres Zimmer Tapered Screw-Vent® 3,5mmD Green [1786]
Core3dcentres	1782	Zimmer	Core3dcentres Zimmer Tapered Screw-Vent® 4,5mmD Violet [1782]
Core3dcentres	2057	Zimmer	Core3dcentres Zimmer Tapered Screw-Vent® 5,7mmD Yellow [2057]

ELOS MEDTECH PINOL A/S

Gothenburg, Sweden | <http://elosmedtech.com> | 46 510-48 43 60

Scan Body Manufacturer	Scan Body ID	Implant Manufacturer	Scan Body / Implant Type
Elos Medtech Pinol A/S	IO 7B-A	Biomet 3i	Biomet 3i™ Certain® 3.25 - IO 7B-A
Elos Medtech Pinol A/S	IO 7B-B	Biomet 3i	Biomet 3i™ Certain® 4.1/5.0/6.0 - IO 7B-B
Elos Medtech Pinol A/S	IO 7A-A	Biomet 3i	Biomet 3i™ External Hex 3.25 - IO 7A-A
Elos Medtech Pinol A/S	IO 7A-B	Biomet 3i	Biomet 3i™ External Hex 4.1/5.0/6.0 - IO 7A-B
Elos Medtech Pinol A/S	IO 9A-A	Camlog Biotechnologies AG	Camlog® Implant System 3.3 - IO 9A-A
Elos Medtech Pinol A/S	IO 9A-B	Camlog Biotechnologies AG	Camlog® Implant System 3.8 - IO 9A-B
Elos Medtech Pinol A/S	IO 9A-C	Camlog Biotechnologies AG	Camlog® Implant System 4.3 - IO 9A-C
Elos Medtech Pinol A/S	IO 9A-D	Camlog Biotechnologies AG	Camlog® Implant System 5.0/6.0 - IO 9A-D
Elos Medtech Pinol A/S	IO 8B-A SA	Dentsply Implants IH	Dentsply Ankylos® - IO 8B-A SA
Elos Medtech Pinol A/S	IO 8C-A	Dentsply Implants IH	Dentsply Ankylos® Balance Base - IO 8C-A
Elos Medtech Pinol A/S	IO 3B-A SA	Dentsply Implants IH	Dentsply AstraTech EV 3.0 - IO 3B-A SA
Elos Medtech Pinol A/S	IO 3B-B SA	Dentsply Implants IH	Dentsply AstraTech EV 3.6 - IO 3B-B SA
Elos Medtech Pinol A/S	IO 3B-C SA	Dentsply Implants IH	Dentsply AstraTech EV 4.2 - IO 3B-C SA
Elos Medtech Pinol A/S	IO 3C-A SA	Dentsply Implants IH	Dentsply AstraTech EV 4.2 Profile - IO 3C-A SA
Elos Medtech Pinol A/S	IO 3B-D SA	Dentsply Implants IH	Dentsply AstraTech EV 4.8 - IO 3B-D SA

Scan Body Manufacturer	Scan Body ID	Implant Manufacturer	Scan Body / Implant Type
Elos Medtech Pinol A/S	IO 3C-B SA	Dentsply Implants IH	Dentsply AstraTech EV 4.8 Profile - IO 3C-B SA
Elos Medtech Pinol A/S	IO 3B-E SA	Dentsply Implants IH	Dentsply AstraTech EV 5.4 - IO 3B-E SA
Elos Medtech Pinol A/S	IO 3A-A SA	Dentsply Implants IH	Dentsply AstraTech™ TX 3.0 - IO 3A-A SA
Elos Medtech Pinol A/S	IO 3A-B SA	Dentsply Implants IH	Dentsply AstraTech™ TX 3.5/4.0 - IO 3A-B SA
Elos Medtech Pinol A/S	IO 3A-C SA	Dentsply Implants IH	Dentsply AstraTech™ TX 4.5/5.0 - IO 3A-C SA
Elos Medtech Pinol A/S	IO 8A-A	Dentsply Implants IH	Dentsply Xive® 3.0 - IO 8A-A
Elos Medtech Pinol A/S	IO 8A-B	Dentsply Implants IH	Dentsply Xive® 3.4 - IO 8A-B
Elos Medtech Pinol A/S	IO 8A-C	Dentsply Implants IH	Dentsply Xive® 3.8 - IO 8A-C
Elos Medtech Pinol A/S	IO 8A-D	Dentsply Implants IH	Dentsply Xive® 4.5/5.5 - IO 8A-D
Elos Medtech Pinol A/S	IO 1B-A	NEOSS	Neoss™ Access - IO 1B-A
Elos Medtech Pinol A/S	IO 1A-B	NEOSS	Neoss™ Ø3.25 - IO 1A-B
Elos Medtech Pinol A/S	IO 1A-A	NEOSS	Neoss™ Ø3.5-5.5 - IO 1A-A
Elos Medtech Pinol A/S	IO 6A-A	Nobel Biocare	Brånemark System® NP - IO 6A-A
Elos Medtech Pinol A/S	IO 6A-B	Nobel Biocare	Brånemark System® RP - IO 6A-B
Elos Medtech Pinol A/S	IO 6A-C	Nobel Biocare	Brånemark System® WP - IO 6A-C
Elos Medtech Pinol A/S	IO 2B-D SA	Nobel Biocare	Nobel Biocare® CC 3.0 - IO 2B-D SA
Elos Medtech Pinol A/S	IO 2B-A SA	Nobel Biocare	Nobel Biocare® CC 3.5 NP - IO 2B-A SA
Elos Medtech Pinol A/S	IO 2B-B SA	Nobel Biocare	Nobel Biocare® CC 4.3/5.0 RP - IO 2B-B SA
Elos Medtech Pinol A/S	IO 2B-C SA	Nobel Biocare	Nobel Biocare® CC WP - IO 2B-C SA
Elos Medtech Pinol A/S	IO 2A-A	Nobel Biocare	NobelReplace® 3.5 NP - IO 2A-A
Elos Medtech Pinol A/S	IO 2A-B	Nobel Biocare	NobelReplace® 4.3 RP - IO 2A-B
Elos Medtech Pinol A/S	IO 2A-C	Nobel Biocare	NobelReplace® 5.0 WP - IO 2A-C
Elos Medtech Pinol A/S	IO 2A-D	Nobel Biocare	NobelReplace® 6.0 WP - IO 2A-D
Elos Medtech Pinol A/S	IO 4A-A SA	Straumann	Straumann® Bone Level™ 3.3 NC - IO 4A-A SA
Elos Medtech Pinol A/S	IO 4A-B SA	Straumann	Straumann® Bone Level™ 4.1/4.8 RC - IO 4A-B SA
Elos Medtech Pinol A/S	IO 4C-A	Straumann	Straumann® NNC - IO 4C-A
Elos Medtech Pinol A/S	IO 4B-A	Straumann	Straumann® Standard RN - IO 4B-A
Elos Medtech Pinol A/S	IO 4B-A SA	Straumann	Straumann® Standard RN - IO 4B-A SA
Elos Medtech Pinol A/S	IO 4B-B	Straumann	Straumann® Standard WN - IO 4B-B
Elos Medtech Pinol A/S	IO 5A-A	Zimmer Dental	Zimmer® Screw-vent 3.5 - IO 5A-A
Elos Medtech Pinol A/S	IO 5A-B	Zimmer Dental	Zimmer® Screw-vent 4.5 - IO 5A-B
Elos Medtech Pinol A/S	IO 5A-C	Zimmer Dental	Zimmer® Screw-vent 5.7 - IO 5A-C
Elos Medtech Pinol A/S	IO 6A-A	Nobel Biocare	Brånemark System® NP
Elos Medtech Pinol A/S	IO 6A-B	Nobel Biocare	Brånemark System® RP
Elos Medtech Pinol A/S	IO 6A-C	Nobel Biocare	Brånemark System® WP

Scan Body Manufacturer	Scan Body ID	Implant Manufacturer	Scan Body / Implant Type
Glidewell	70-1047-COM0021	Biomet 3i	Glidewell Biomet 3i Certain Internal 3.4 mm [70-1047-COM0021]10059
Glidewell	70-1047-COM0022	Biomet 3i	Glidewell Biomet 3i Certain Internal 4.1 mm [70-1047-COM0022]
Glidewell	70-1047-COM0023	Biomet 3i	Glidewell Biomet 3i Certain Internal 5.0 mm [70-1047-COM0023]
Glidewell	70-1047-COM0024	Biomet 3i	Glidewell Biomet 3i Certain Internal 6.0 mm [70-1047-COM0024]
Glidewell	70-1135-CAM0103	CAMLOG	Glidewell CAMLOG SCREW-LINE 3.3 mm [70-1135-CAM0103]
Glidewell	70-1135-CAM0105	CAMLOG	Glidewell CAMLOG SCREW-LINE 3.8 mm [70-1135-CAM0105]
Glidewell	70-1135-CAM0107	CAMLOG	Glidewell CAMLOG SCREW-LINE 4.3 mm [70-1135-CAM0107]
Glidewell	70-1135-CAM0109	CAMLOG	Glidewell CAMLOG SCREW-LINE 5.0 mm [70-1135-CAM0109]

GLIDEWELL LABORATORIES (continued)

Newport Beach, CA | www.glidewell dental.com | 800-854-7256 USA | 888-278-0414 Canada

Scan Body Manufacturer	Scan Body ID	Implant Manufacturer	Scan Body / Implant Type
Glidewell	70-1135-CAM0111	CAMLOG	Glidewell CAMLOG SCREW-LINE 6.0 mm [70-1135-CAM0111]
Glidewell	70-1048-COM0028	Dentsply	Glidewell Dentsply Astra OsseoSpeed 3.0 [70-1048-COM0028]
Glidewell	70-1048-COM0029	Dentsply	Glidewell Dentsply Astra OsseoSpeed 3.5/4.0 [70-1048-COM0029]
Glidewell	70-1048-COM0030	Dentsply	Glidewell Dentsply Astra OsseoSpeed 4.5/5.0 [70-1048-COM0030]
Glidewell	70-1055-COM0004	Keystone Dental	Glidewell Keystone PrimaConnex RD [70-1055-COM0004]
Glidewell	70-1055-COM0005	Keystone Dental	Glidewell Keystone PrimaConnex SD [70-1055-COM0005]
Glidewell	70-1055-COM0006	Keystone Dental	Glidewell Keystone PrimaConnex WD [70-1055-COM0006]
Glidewell	70-1056-COM0002	NEOSS	Glidewell NEOSS 4.0 mm [70-1056-COM0002]
Glidewell	70-1049-COM0008	Nobel Biocare	Glidewell Nobel Brånemark System RP [70-1049-COM0008]
Glidewell	70-1050-COM0005	Nobel Biocare	Glidewell Nobel NobelActive NP [70-1050-COM0005]
Glidewell	70-1050-COM0006	Nobel Biocare	Glidewell Nobel NobelActive RP [70-1050-COM0006]
Glidewell	70-1051-COM0011	Nobel Biocare	Glidewell Nobel NobelReplace NP [70-1051-COM0011]
Glidewell	70-1051-COM0010	Nobel Biocare	Glidewell Nobel NobelReplace Ø6 [70-1051-COM0014]
Glidewell	70-1051-COM0012	Nobel Biocare	Glidewell Nobel NobelReplace RP [70-1051-COM0012]
Glidewell	70-1051-COM0013	Nobel Biocare	Glidewell Nobel NobelReplace WP [70-1051-COM0013]
Glidewell	70-1073-PRA0003	Prismatik DentalCraft	Glidewell Prismatik Inclusive 3.5mm [70-1073-PRA0003]
Glidewell	70-1073-PRA0007	Prismatik DentalCraft	Glidewell Prismatik Inclusive 4.5mm [70-1073-PRA0007]
Glidewell	70-1073-PRA0007	Prismatik DentalCraft	Glidewell Prismatik Inclusive 5.2mm [70-1073-PRA0007]
Glidewell	70-1052-COM0009	Straumann	Glidewell Straumann Bone Level NC [70-1052-COM0009]
Glidewell	70-1052-COM0010	Straumann	Glidewell Straumann Bone Level RC [70-1052-COM0010]
Glidewell	70-1054-COM0013	Zimmer	Glidewell Zimmer Screw-Vent 3.5 mm [70-1054-COM0013]
Glidewell	70-1054-COM0014	Zimmer	Glidewell Zimmer Screw-Vent 4.5 mm [70-1054-COM0014]
Glidewell	70-1054-COM0015	Zimmer	Glidewell Zimmer Screw-Vent 5.7 mm [70-1054-COM0015]

MEDENTIKA® – IPS – IMPLANT SYSTEMS

Hügelsheim, Germany | www.medentika.de | +49 (0)7229 69912-0

Scan Body Manufacturer	Scan Body ID	Implant Manufacturer	Scan Body / Implant Type
Medentika®	1-09-03	Medentika®	Microcone® Implant NI; D 3,0mm; L 11mm
Medentika®	1-09-03	Medentika®	Microcone® Implant NI; D 3,0mm; L 13mm
Medentika®	1-09-03	Medentika®	Microcone® Implant NI; D 3,0mm; L 15mm
Medentika®	2-09-10	Medentika®	Microcone® Implant RI; D 3,5mm; L 8mm
Medentika®	2-09-10	Medentika®	Microcone® Implant RI; D 3,5mm; L 9mm
Medentika®	2-09-10	Medentika®	Microcone® Implant RI; D 3,5mm; L 11mm
Medentika®	2-09-10	Medentika®	Microcone® Implant RI; D 3,5mm; L 13mm
Medentika®	2-09-10	Medentika®	Microcone® Implant RI; D 3,5mm; L 15mm
Medentika®	2-09-10	Medentika®	Microcone® Implant RI; D 4,0mm; L 6,5mm
Medentika®	2-09-10	Medentika®	Microcone® Implant RI; D 4,0mm; L 8mm
Medentika®	2-09-10	Medentika®	Microcone® Implant RI; D 4,0mm; L 9mm
Medentika®	2-09-10	Medentika®	Microcone® Implant RI; D 4,0mm; L 11mm
Medentika®	2-09-10	Medentika®	Microcone® Implant RI; D 4,0mm; L 13mm
Medentika®	2-09-10	Medentika®	Microcone® Implant RI; D 4,0mm; L 15mm
Medentika®	2-09-10	Medentika®	Microcone® Implant RI; D 4,5mm; L 6,5mm
Medentika®	2-09-10	Medentika®	Microcone® Implant RI; D 4,5mm; L 8mm
Medentika®	2-09-10	Medentika®	Microcone® Implant RI; D 4,5mm; L 9mm
Medentika®	2-09-10	Medentika®	Microcone® Implant RI; D 4,5mm; L 11mm
Medentika®	2-09-10	Medentika®	Microcone® Implant RI; D 4,5mm; L 13mm

MEDENTIKA® – IPS – IMPLANT SYSTEMS (continued)

Hügelsheim, Germany | www.medentika.de | +49 (0)7229 69912-0

Scan Body Manufacturer	Scan Body ID	Implant Manufacturer	Scan Body / Implant Type
Medentika®	2-09-10	Medentika®	Microcone® Implant RI; D 4,5mm; L 15mm
Medentika®	2-09-10	Medentika®	Microcone® Implant RI; D 5,0mm; L 6,5mm
Medentika®	2-09-10	Medentika®	Microcone® Implant RI; D 5,0mm; L 8mm
Medentika®	2-09-10	Medentika®	Microcone® Implant RI; D 5,0mm; L 9mm
Medentika®	2-09-10	Medentika®	Microcone® Implant RI; D 5,0mm; L 11mm
Medentika®	2-09-10	Medentika®	Microcone® Implant RI; D 5,0mm; L 13mm
Medentika®	2-09-10	Medentika®	Microcone® Implant RI; D 5,0mm; L 15mm
Medentika®	2-09-10	Medentika®	Microcone® Implant conical RI; D 4,5/3,5; L 6,5mm
Medentika®	2-09-10	Medentika®	Microcone® Implant conical RI; D 4,5/3,5; L 9mm
Medentika®	2-09-10	Medentika®	Microcone® Implant conical RI; D 4,5/3,5; L 11mm
Medentika®	2-09-10	Medentika®	Microcone® Implant conical RI; D 4,5/3,5; L 13mm
Medentika®	2-09-10	Medentika®	Quattrocone® Implant RI; D 3,5mm; L 9mm
Medentika®	2-09-10	Medentika®	Quattrocone® Implant RI; D 3,5mm; L 11mm
Medentika®	2-09-10	Medentika®	Quattrocone® Implant RI; D 3,5mm; L 13mm
Medentika®	2-09-10	Medentika®	Quattrocone® Implant RI; D 3,5mm; L 15mm
Medentika®	2-09-10	Medentika®	Quattrocone® Implant RI; D 4,3mm; L 7mm
Medentika®	2-09-10	Medentika®	Quattrocone® Implant RI; D 4,3mm; L 9mm
Medentika®	2-09-10	Medentika®	Quattrocone® Implant RI; D 4,3mm; L 11mm
Medentika®	2-09-10	Medentika®	Quattrocone® Implant RI; D 4,3mm; L 13mm
Medentika®	2-09-10	Medentika®	Quattrocone® Implant RI; D 4,3mm; L 15mm
Medentika®	2-09-10	Medentika®	Quattrocone® Implant RI; D 5,0mm; L 7mm
Medentika®	2-09-10	Medentika®	Quattrocone® Implant RI; D 5,0mm; L 9mm
Medentika®	2-09-10	Medentika®	Quattrocone® Implant RI; D 5,0mm; L 11mm
Medentika®	2-09-10	Medentika®	Quattrocone® Implant RI; D 5,0mm; L 13mm
Medentika®	2-09-10	Medentika®	Quattrocone® Implant RI; D 5,0mm; L 15mm
Medentika®	4-09-01	Medentika®	Quattrocone30 Implant AI; D 4,3mm; L 9mm
Medentika®	4-09-01	Medentika®	Quattrocone30 Implant AI; D 4,3mm; L 11mm
Medentika®	4-09-01	Medentika®	Quattrocone30 Implant AI; D 4,3mm; L 13mm
Medentika®	4-09-01	Medentika®	Quattrocone30 Implant AI; D 4,3mm; L 15mm
Medentika®	C 1400	Medentika®	Procone Implant D 3,3mm; L 11mm
Medentika®	C 1400	Medentika®	Procone Implant D 3,3mm; L 13mm
Medentika®	C 1410	Medentika®	Procone Implant D 3,8mm; L 9mm
Medentika®	C 1410	Medentika®	Procone Implant D 3,8mm; L 11mm
Medentika®	C 1410	Medentika®	Procone Implant D 3,8mm; L 13mm
Medentika®	C 1420	Medentika®	Procone Implant D 4,3mm; L 9mm
Medentika®	C 1420	Medentika®	Procone Implant D 4,3mm; L 11mm
Medentika®	C 1420	Medentika®	Procone Implant D 4,3mm; L 13mm
Medentika®	C 1430	Medentika®	Procone Implant D 5,0mm; L 9mm
Medentika®	C 1430	Medentika®	Procone Implant D 5,0mm; L 11mm
Medentika®	C 1430	Medentika®	Procone Implant D 5,0mm; L 13mm

MEDENTIKA® – MPS – MULTI PLATFORM SYSTEMS

Hügelsheim, Germany | www.medentika.de | +49 (0)7229 69912-0

Scan Body Manufacturer	Scan Body ID	Implant Manufacturer	Scan Body / Implant Type
B-Series			
Medentika®	B1400	Bredent Medical®	Bredent Sky® 3,5-5,5
BS-Series			
Medentika®	BS 1400	BEGO Implant Systems®	BEGO Semados® S / RI / RS / RSX 3,25/3,75
Medentika®	BS 1400	BEGO Implant Systems®	BEGO Semados® S / RI / RS / RSX 4,1
Medentika®	BS 1400	BEGO Implant Systems®	BEGO Semados® S / RI / RS / RSX 4,5
Medentika®	BS 1400	BEGO Implant Systems®	BEGO Semados® S / RI / RS / RSX 5,5
C-Series			
Medentika®	C 1400	Altatec®	Altatec Camlog® 3,3
Medentika®	C 1410	Altatec®	Altatec Camlog® 3,8
Medentika®	C 1420	Altatec®	Altatec Camlog® 4,3
Medentika®	C 1430	Altatec®	Altatec Camlog® 5,0
Medentika®	C 1430	Altatec®	Altatec Camlog® 6,0
CX-Series			
Medentika®	CX 1400	Medentis Medical®	Medentis Medical ICX® 3,75-4,8
D-Series			
Medentika®	D 1400	Altatec®	Altatec Conelog® 3,3
Medentika®	D 1410	Altatec®	Altatec Conelog® 3,8/4,3
E-Series			
Medentika®	E 1400	Nobel Biocare	Nobel Biocare NobelReplace Tapered® 3,5 NP
Medentika®	E 1410	Nobel Biocare	Nobel Biocare NobelReplace Tapered® 4,3 RP
Medentika®	E 1420	Nobel Biocare	Nobel Biocare NobelReplace Tapered® 5,0 WP
Medentika®	E 1430	Nobel Biocare	Nobel Biocare NobelReplace Tapered® 6,0
EV-Series			
Medentika®	EV 1400	Dentsply Implants®	Dentsply Implants ASTRA TECH OsseoSpeed® EV 3,0
Medentika®	EV 1410	Dentsply Implants®	Dentsply Implants ASTRA TECH OsseoSpeed® EV 3,6
Medentika®	EV 1420	Dentsply Implants®	Dentsply Implants ASTRA TECH OsseoSpeed® EV 4,2
Medentika®	EV 1430	Dentsply Implants®	Dentsply Implants ASTRA TECH OsseoSpeed® EV 4,8
Medentika®	EV 1440	Dentsply Implants®	Dentsply Implants ASTRA TECH OsseoSpeed® EV 5,4
F-Series			
Medentika®	F 1420	Nobel Biocare®	Nobel Biocare NobelReplace Conical® 3,0
Medentika®	F 1400	Nobel Biocare®	Nobel Biocare NobelReplace Conical® 3,5 NP
Medentika®	F 1410	Nobel Biocare®	Nobel Biocare NobelReplace Conical® 4,3 5,0 RP
H-Series			
Medentika®	H 1400	Biomet 3i®	Biomet 3i Certain® 3,4
Medentika®	H 1410	Biomet 3i®	Biomet 3i Certain® 4,1
Medentika®	H 1410	Biomet 3i®	Biomet 3i Certain® 5,0
I-Series			
Medentika®	I 1400	Biomet 3i®	Biomet 3i Osseotite® "External Hex" 3,4
Medentika®	I 1410	Biomet 3i®	Biomet 3i Osseotite® "External Hex" 4,1
Medentika®	I 1410	Biomet 3i®	Biomet 3i Osseotite® "External Hex" 5,0
K-Series			
Medentika®	K 1400	Nobel Biocare®	Nobel Biocare Branemark® 3,5 NP
Medentika®	K 1410	Nobel Biocare®	Nobel Biocare Branemark® 4,1 RP
Medentika®	K 1420	Nobel Biocare®	Nobel Biocare Branemark® 5,1 WP

MEDENTIKA® – MPS – MULTI PLATFORM SYSTEMS

Hügelsheim, Germany | www.medentika.de | +49 (0)7229 69912-0

Scan Body Manufacturer	Scan Body ID	Implant Manufacturer	Scan Body / Implant Type
L-Series			
Medentika®	L 1400	Straumann®	Straumann Bone Level® 3,3 NC
Medentika®	L 1410	Straumann®	Straumann Bone Level® 4,1/4,8 RC
N-Series			
Medentika®	N 1430	Straumann®	Straumann Soft Tissue Level® 3,5 NNC
Medentika®	N 1410	Straumann®	Straumann Soft Tissue Level® 4,8 RN
Medentika®	N 1420	Straumann®	Straumann Soft Tissue Level® 6,5 WN
Medentika®	N 1400	Straumann®	Straumann Soft Tissue Level® 3,5 NN
R-Series			
Medentika®	R 1400	Zimmer Dental®	Zimmer Tapered Screw-Vent® 3,5
Medentika®	R 1410	Zimmer Dental®	Zimmer Tapered Screw-Vent® 4,5
Medentika®	R 1420	Zimmer Dental®	Zimmer Tapered Screw-Vent® 5,7
S-Series			
Medentika®	S 1430	Dentsply Implants®	Dentsply Implants ASTRA TECH OsseoSpeed® TX 3,0
Medentika®	S 1400	Dentsply Implants®	Dentsply Implants ASTRA TECH OsseoSpeed® TX 3,5/4,0
Medentika®	S 1420	Dentsply Implants®	Dentsply Implants ASTRA TECH OsseoSpeed® TX 4,5/5,0
T-Series			
Medentika®	T 1400	Dentsply Implants®	Dentsply Implants XiVE® 3,4
Medentika®	T 1405	Dentsply Implants®	Dentsply Implants XiVE® 3,8
Medentika®	T 1410	Dentsply Implants®	Dentsply Implants XiVE® 4,5
Medentika®	T 1410	Dentsply Implants®	Dentsply Implants XiVE® 5,5
Y-Series			
Medentika®	Y 1400	Dentsply Implants®	Dentsply Implants ANKYLOS® 3,5-7,0

ZIMMER DENTAL

Carlsbad, CA | www.zimmerdental.com | 760-929-4300 or 800-854-7019 USA

Scan Body Manufacturer	Scan Body ID	Implant Manufacturer	Scan Body / Implant Type
Zimmer	PSA3SCAN	Zimmer	Zimmer Clinical Scan Body 3.5
Zimmer	PSA4SCAN	Zimmer	Zimmer Clinical Scan Body 4.5
Zimmer	PSA5SCAN	Zimmer	Zimmer Clinical Scan Body 5.7



Align Technology, Inc.
2560 Orchard Parkway
San Jose, California 95131

INVISALIGN, CLINCHECK, and ITERO, among others, are trademarks and / or service marks of Align Technology, Inc. or one of its subsidiaries or affiliated companies and may be registered in the U.S. and / or other countries.

© 2017 Align Technology, Inc. M20298 Rev D