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분류	pH	제조사
Birex Quat Wipes	7.6	Biotrol Intl.
CaviCide AF	12.7	Metrex
CaviCide CaviWipe	12.5	Metrex
CaviCide 1 CaviWipe 1	12.5	Metrex
Clorox Healthcare Hydrogen Peroxide Cleaner Disinfecting Liquid	2-3	Clorox Healthcare
Clorox Healthcare Hydrogen Peroxide Cleaner Disinfectant Wipes	2-3	Clorox Healthcare
Opti-Cide 3 Liquid	7.6	Biotrol Intl.
Opti Cide 3 Wipes	7.6	Biotrol Intl.
OPTIM 33TB Liquid	2.5-3.5	SciCan Inc.
OPTIM 33TB Wipes	2.5-3.5	SciCan Inc.
ProSpray	10	Certol
ProSpray wipes		
Webcol Alcohol Prep Pads	7	Medtronic

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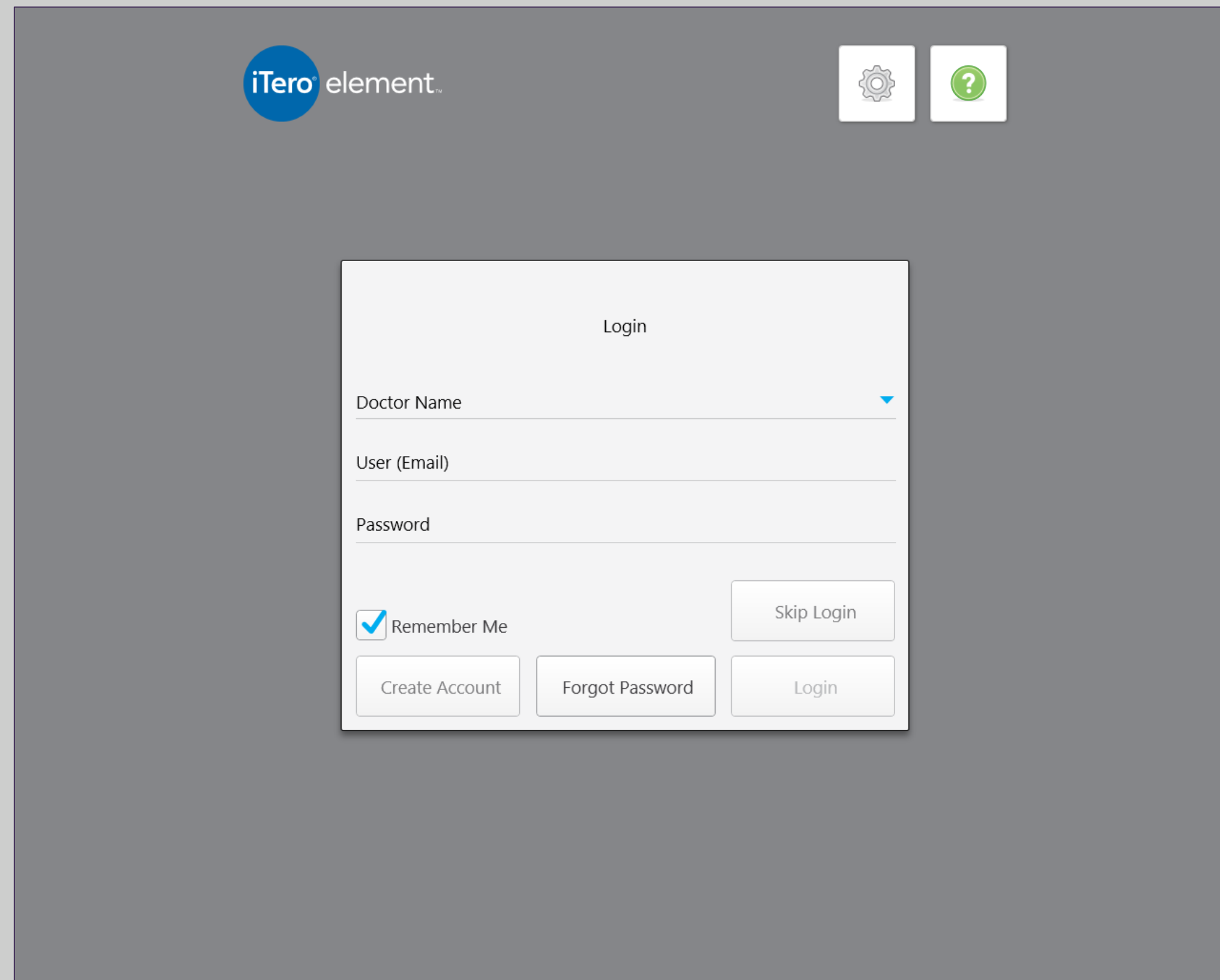
Make It Mine / Registering the System

To register the iTero Element system, have the following information available*

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The image shows a login interface for iTero element. At the top left is the iTero element logo. To its right are two icons: a gear for settings and a question mark for help. The main login form is centered and titled "Login". It contains three input fields: "Doctor Name" with a dropdown arrow, "User (Email)", and "Password". Below the "Password" field is a "Remember Me" checkbox, which is checked. To the right of the "Remember Me" checkbox is a "Skip Login" button. At the bottom of the form are three buttons: "Create Account", "Forgot Password", and "Login".

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Dr. Align, Test


In Progress

ID	Patient Name	Chart Number	Scan Date	Doctor	Case Type	Status
	Test, Bridge			Dr. Align, Test	Restorative	Rx Created
	Tes, Tes			Dr. Align, Test	Restorative	Rx Created
	Smith, Paige	8594	07/31/2015 09:40:45	Dr. Align, Test	Invisalign	Scanning

Past Orders

Search patients

ID	Patient Name	Chart Number	Scan Date	Doctor	Case Type	Status
8906781	Restorative, Patient		02/01/2016	Dr. Align, Test	Invisalign + i	Completed
8907458	Restorative, Patient		02/01/2016	Dr. Align, Test	iRecord	Completed
8906583	Restorative, Patient		02/01/2016	Dr. Align, Test	Quadrant	iTero Modeling
8906919	test, test		02/01/2016	Dr. Align, Test	Reference Mc	Lab Review
8907174	Restorative, Patient		02/01/2016	Dr. Align, Test	Reference Mc	Inactive/Cancelled

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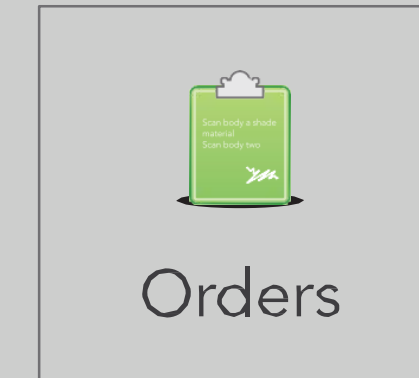
Add Rx

Invisalign Outcome Simulator

9358875	Test, Test		05/05/2016	Dr. Test, Test	iRecord	Completed
---------	------------	--	------------	----------------	---------	-----------

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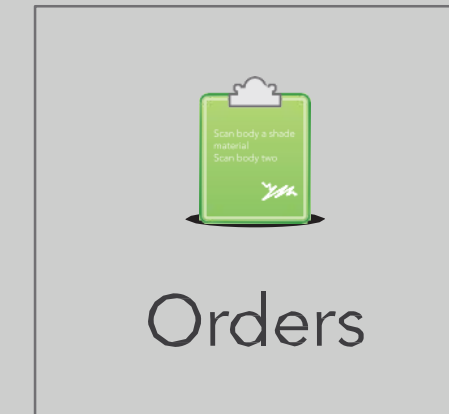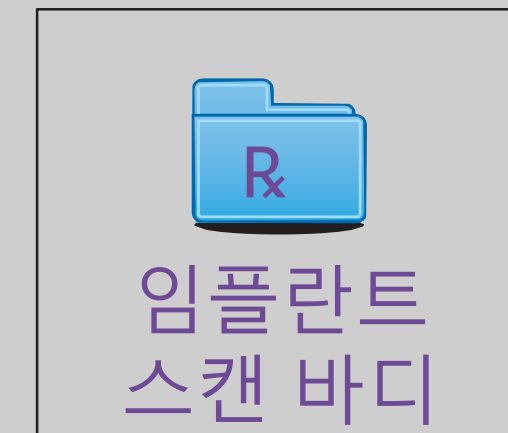
Brackets Preset

Ship To:

Patient already exists - additional scans will be added to existing patient's records.

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
Occlusogram

The screenshot shows a dental software interface titled 'Viewer' with 'Dr. Demo, iGo' in the top right. The interface is divided into several panels. The top-left panel shows a top-down view of the upper and lower dental arches, with a red area labeled '1' indicating a specific region. The top-right panel shows a 3D perspective view of the upper and lower dental arches. The bottom row contains three panels showing different views of the dental model: a side view, a front view, and a back view. A color scale bar is visible in the bottom-right corner of the interface, ranging from 0.0 to 1.0. On the right side of the interface, there is a vertical toolbar with six icons: a blue square, two blue squares, a grid of four blue squares, a color wheel, and a line graph. Yellow arrows point from the Korean labels to these icons. A yellow arrow also points from the '모델 보기 사전설정' label to the top-right panel.

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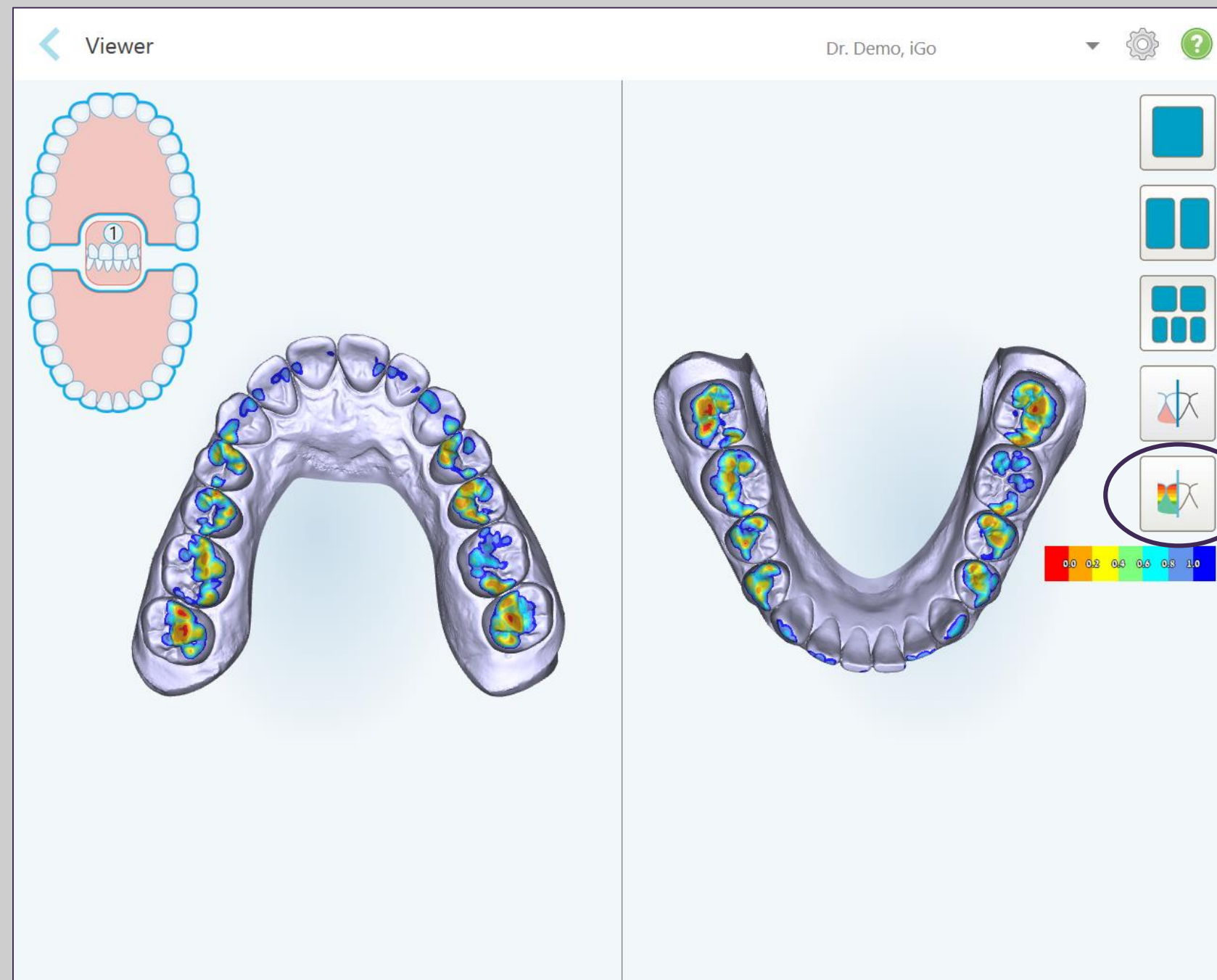
New Scan

Orders

ID	Scan Date	Doctor	Case Type	Status
9358961	05/05/2016	Dr. Test, Test	iRecord	Completed

View Rx Viewer Add Rx

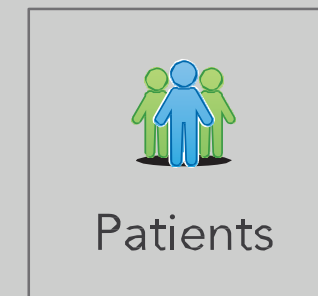Invisalign Outcome Simulator

Orders Dr. Test, Test

In Progress

ID	Patient Name	Chart Number	Scan Date	Doctor	Case Type	Status
9360096	Smith, Jessica		05/05/2016	Dr. Test, Test	iCast	Completed
9358961	Smith-Doe, John		05/05/2016	Dr. Test, Test	iRecord	Completed


Past Orders

Search

ID	Patient Name	Chart Number	Scan Date	Doctor	Case Type	Status
9360096	Smith, Jessica		05/05/2016	Dr. Test, Test	iCast	Completed
9358961	Smith-Doe, John		05/05/2016	Dr. Test, Test	iRecord	Completed
9358875	Test, Test		05/05/2016	Dr. Test, Test	iRecord	Completed

View Rx Viewer Add Rx

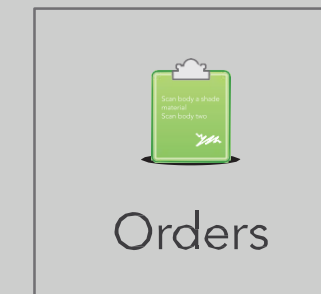Invisalign Outcome Simulator

MYALIGNTECH iTero • OrthoCAD test@testperson.com (0) Profile Logout

Orders iTero Store

Order Information

Order ID: 8965139 First Name: test Company: Cadent Support NJ Demo

Order Code: 0PKZ1X Last Name: test Doctor: Dr. Test, Test

Case Type: Invisalign Chart Number: Acquisition Date: 2016/02/12 Doctor License: 12345

Registration Date: 2016/02/12 State: Completed Modeling Center: Align Technology de Co Tracking Number:

Rx notes:

Actions: Open with IQSim Open File Export (OrthoCAD 3.5 or higher) Image Management Open RX

Order Details

Order Details ID	Item	Quantity	Ship To	Bill To	Due Date	Shipping Date	Delivery Date	Status
12588583	Invisalign	1.00	Cadent Support NJ Demo	Cadent Support NJ Demo	2016/02/12	2016/02/12		Completed

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Invisalign, Ortho doc

In Progress

ID	Patient Name	Chart Number	Scan Date	Doctor	Case Type	Status
	Frink, Prof		09/28/2016 13:55:16	Invisalign, Ortho	Restorative	Scanning
	Hhh, Hhh			Invisalign, Ortho	Restorative	Rx Created
	Test, Test			Invisalign, Ortho	Restorative	Rx Created
	Salazar, Allan		09/23/2016 09:50:05	Invisalign, Ortho	Restorative	Scanning

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MAND

PATIENT: Branch, David

Please check axes and make necessary corrections prior to Progress Assessment

UNDO

REDO

RESET

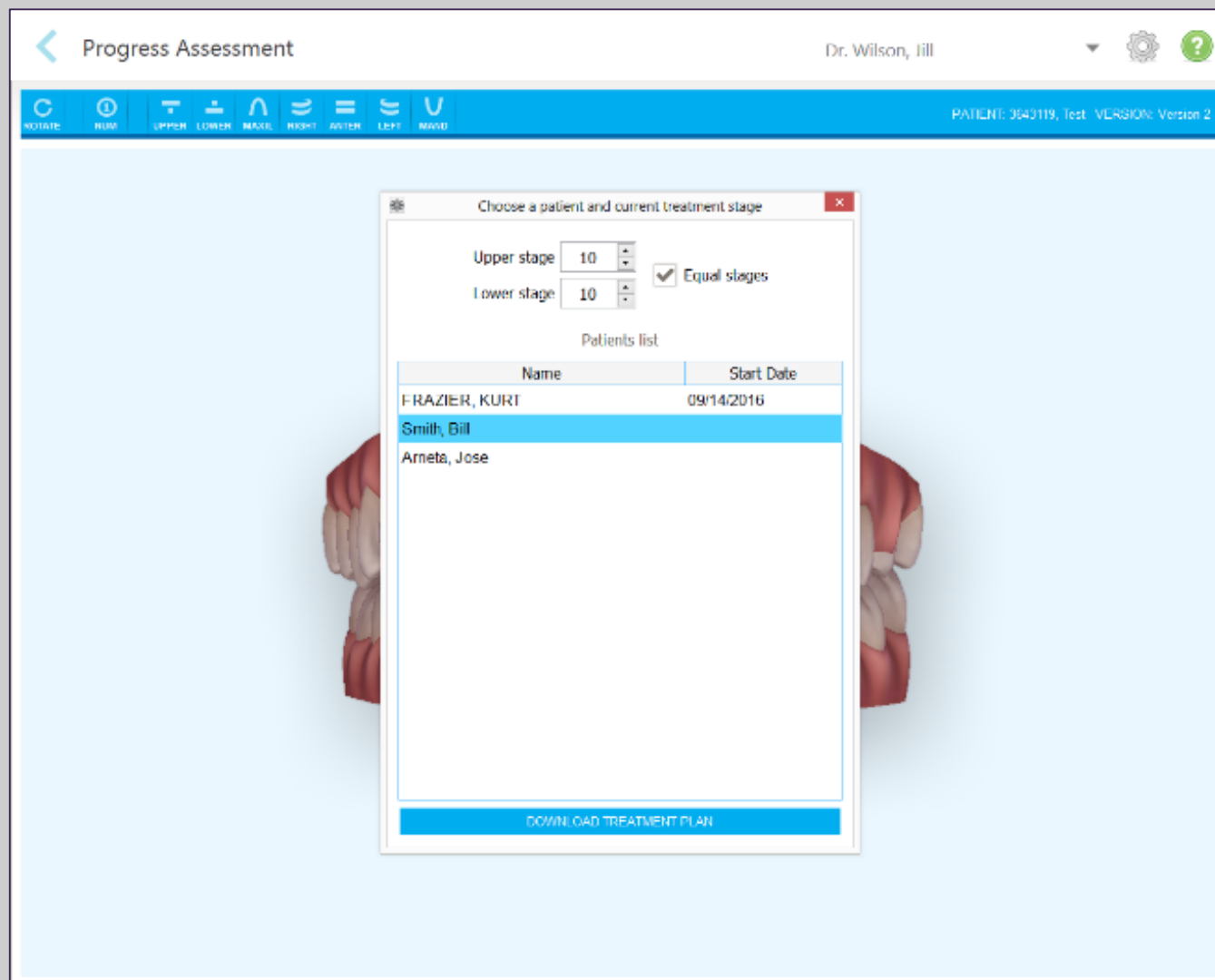
Progress Assessment will start in 7 seconds

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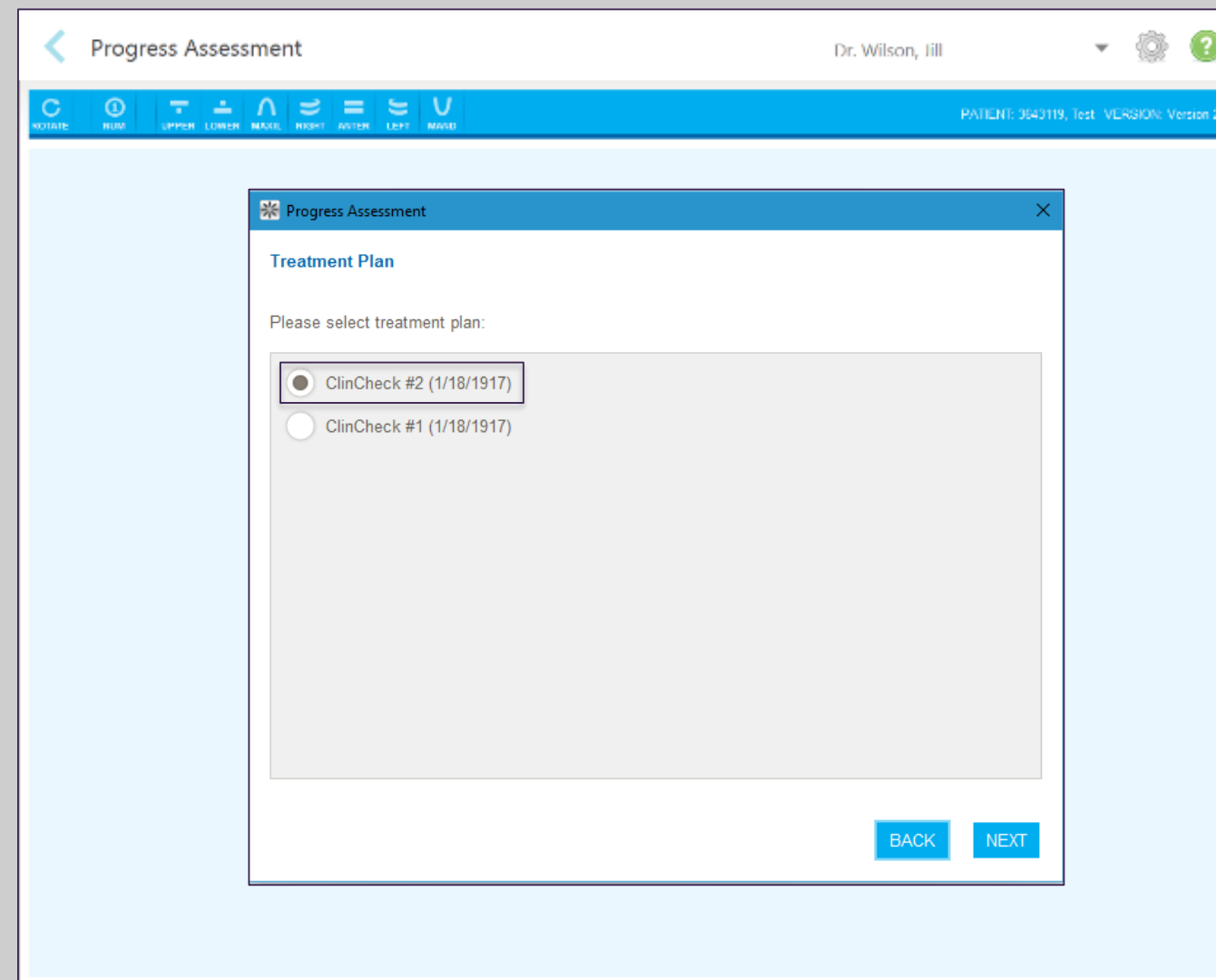
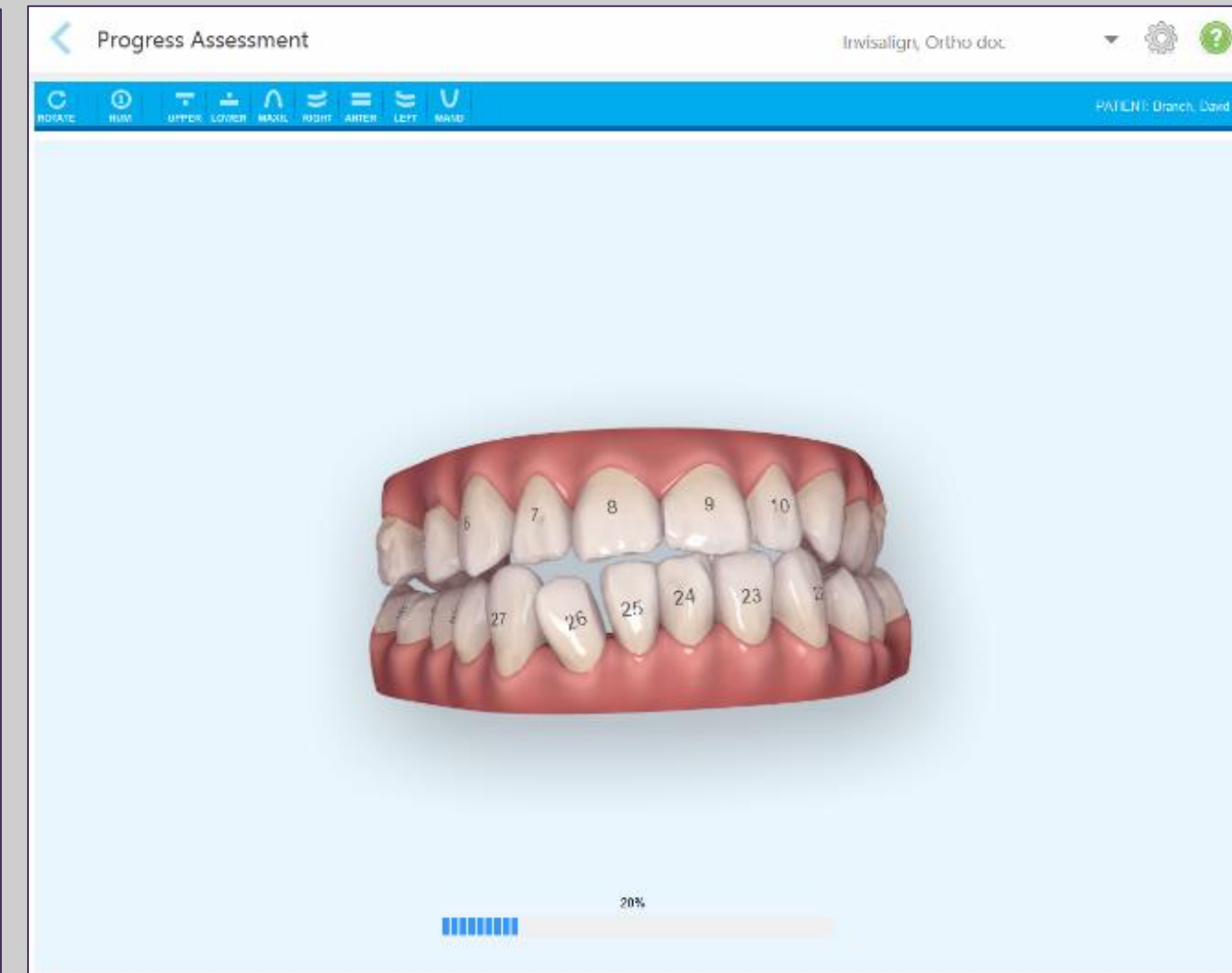
Stage: 28/45

Superimposition

- Teeth from ClinCheck Treatment Plan
- Teeth from new scan

Progress Assessment

- Minor movements not assessed
- Movement tracking clinically to plan
- Movement not tracking clinically to plan
- No significant movement detected
- Movement in opposite direction to plan
- Semi-transparent: Assessment not available

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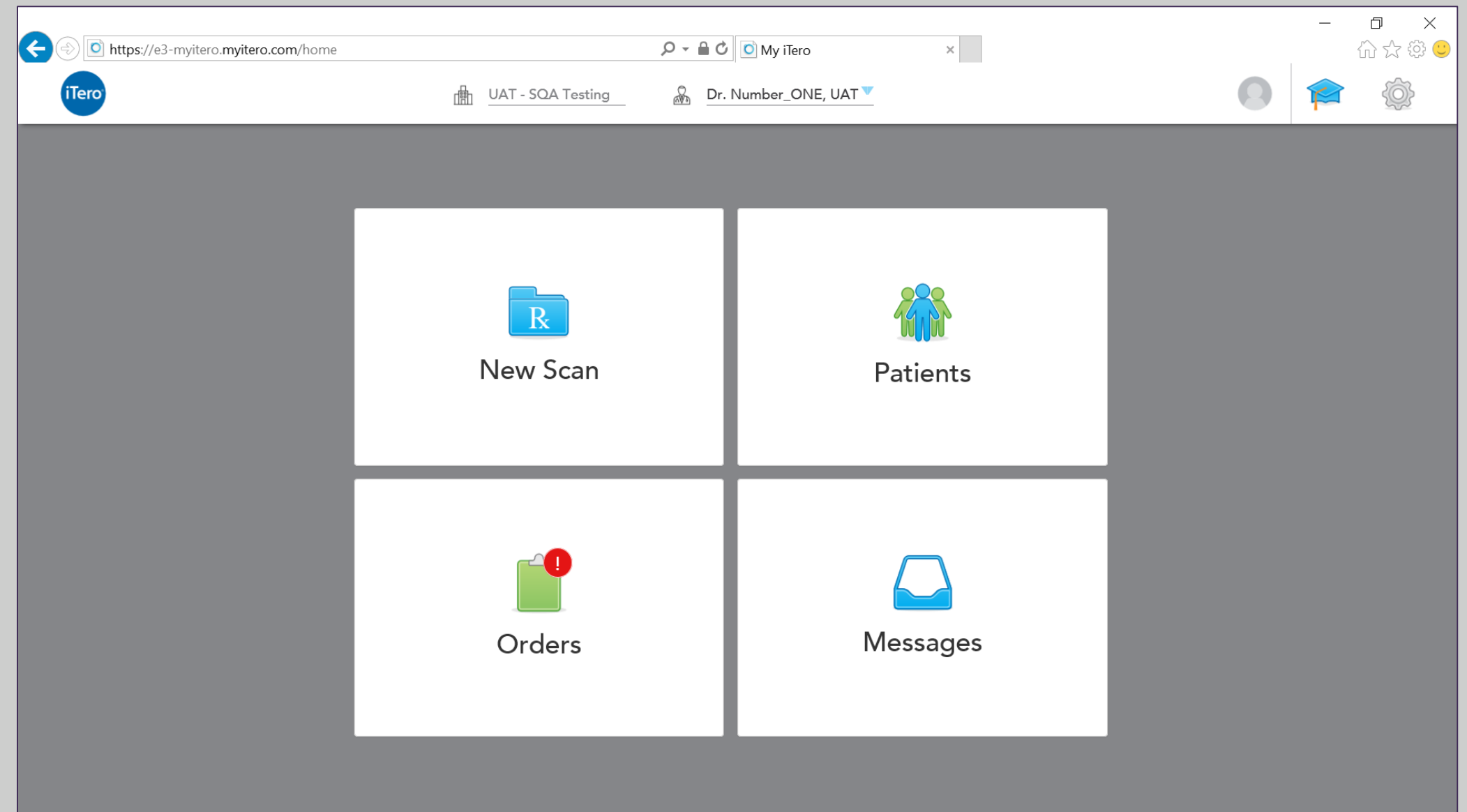
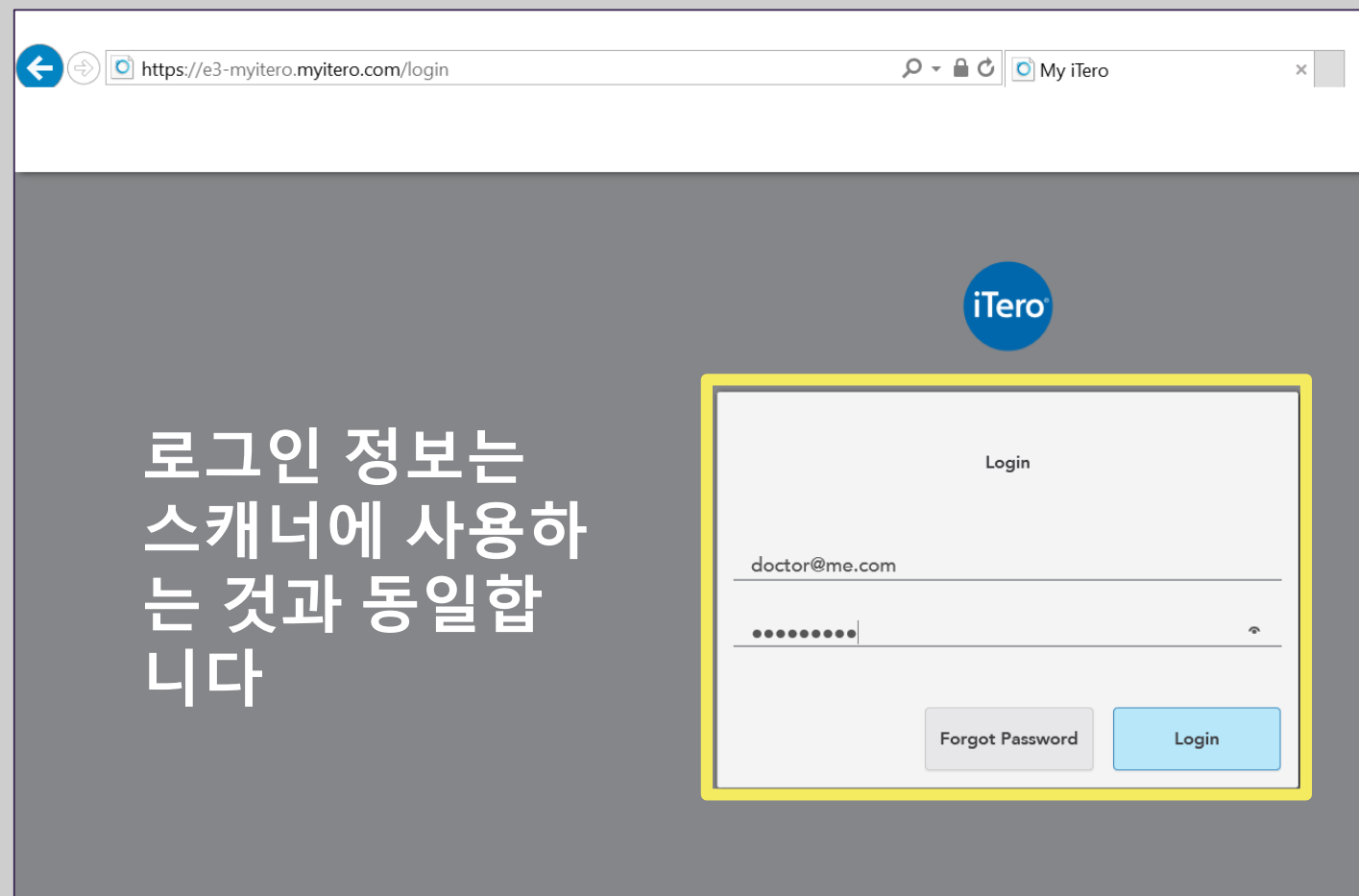
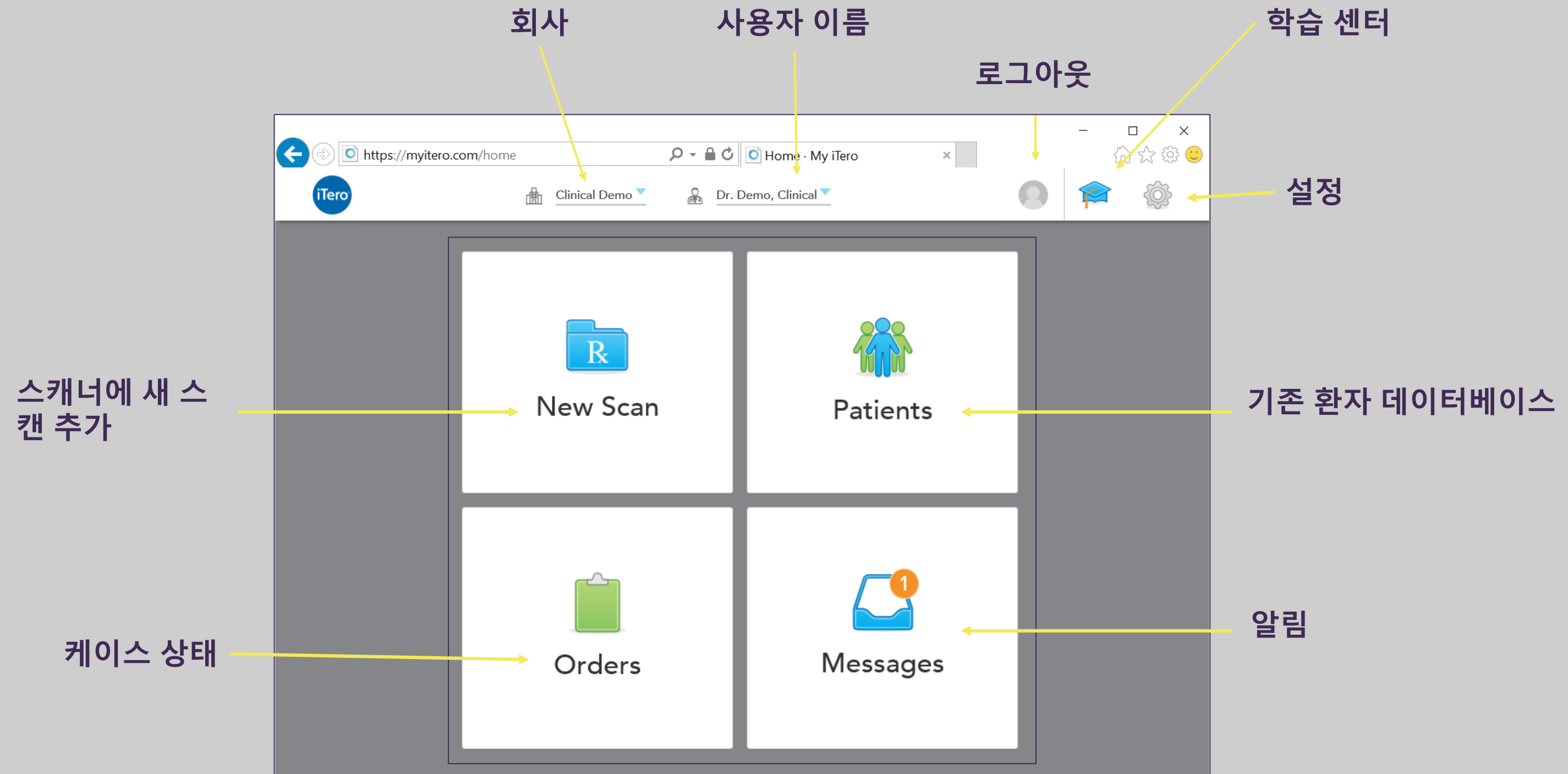
The screenshot displays the MyiTero.com web application interface. On the left, a sidebar shows patient information: Patient Name (Demo, MultiBite), Chart Number (MB0123), and Last Scan (03/28/2017). The main area is titled 'Orders' and contains a table with columns: ID, Scan Date, Case Type, and Status. A single row is visible with ID 11381658, Scan Date 03/28/2017, Case Type iRecord, and Status Completed. Below the table, there are several buttons: View Rx, Change Case Type, Viewer, Export, Invisalign Outcome Simulator, and Invisalign Progress Assessment. A 'New Scan' button is also present in the sidebar. Arrows from the text labels above point to these buttons: 'Rx 열기' points to 'View Rx', '케이스 유형 변경' points to 'Change Case Type', 'Invisalign 결과 시뮬레이터 (Invisalign Outcome Simulator) 열기' points to 'Invisalign Outcome Simulator', 'Invisalign 진행 평가(Invisalign Progress Assessment) 열기' points to 'Invisalign Progress Assessment', '웹 뷰어 (Viewer) 열기' points to 'Viewer', and 'STL 내보내기 (Export)' points to 'Export'.

ID	Scan Date	Case Type	Status
11381658	03/28/2017	iRecord	Completed

View Rx Change Case Type Viewer Export Invisalign Outcome Simulator Invisalign Progress Assessment

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